

Enzymatic solution is NOT approved for use with ophthalmic instruments

Enzymatic Soaking Solution

1. Enzymatic solution is used to soak instruments that are left in the clinic overnight, are heavily soiled, or have lumens.
2. Enzymatic solution can only be used in the soiled utility room.
3. Gloves, gown, and eye protection must be worn while pouring or discarding enzymatic solution.
4. The container must be labeled with the name of the solution, pour date, and expiration date.
5. Solution must be changed daily on open clinic days.
6. Container must remain covered at all times except to place instruments or change solution.



Manufacturer:	Getinge
Product:	Renuzyme Ultra Enzymatic Detergent (1 gal)
Manufacturer Product#:	61301606716
PMM:	112873

**Remember to dilute according to directions for use on the bottle and wear PPE appropriate for the task based on Soiled Instrument Handling training (see tip sheet on the Ambulatory website for review.)*